

### Product Features

- Provides full facial protection
- Reusable frame and shield
- Hassle-free and lightweight
- Sturdy and comfortable to wear
- Non-fogging
- Adjustable frame – one size fits all
- Fits over glasses and masks
- Protects against spray or splatter
- NOT to be used for impact



### Dimensions

**Shield Size:** 13.8"W x 9.25"H  
**Shield Thickness:** .010 mm  
**Frame Size:** 7" diameter, 0.8 oz.

### Materials Used

**Frame:** Polystyrene  
**Shield:** PETG

### Kit Contains

Angel Lite™ Frame  
Angel Lite™ Disposable Shield

### Cleaning & Sterilization Instructions

- Immerse or rinse with soap and warm water to remove any grit
- Dry with a soft cloth (avoid using abrasive cleaners)
- Both the frame and the shield may be disinfected or sterilized using gas, UV, or cold sterilization solutions, which do not leave residue
- DO NOT USE HEAT

### FDA Approved Materials

**Complies with the OSHA Regulation 29HCF 1910.1030**





**281.999.2002**

info@amdrecor.com  
amdrecor.com



## Angel Lite™

The Angel Lite protects your eyes and face, providing full facial protection. It is comfortable, lightweight, and disposable. The head frame is designed for ease of use, single-use, or limited multiuse while the non-fogging shield provides top coverage, airflow, and unobstructed panoramic vision.

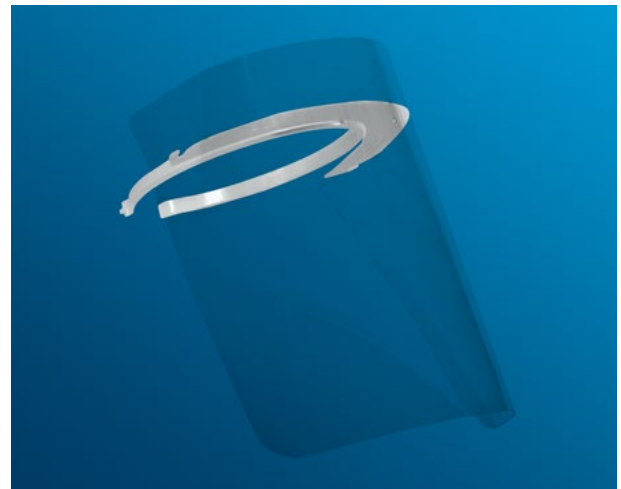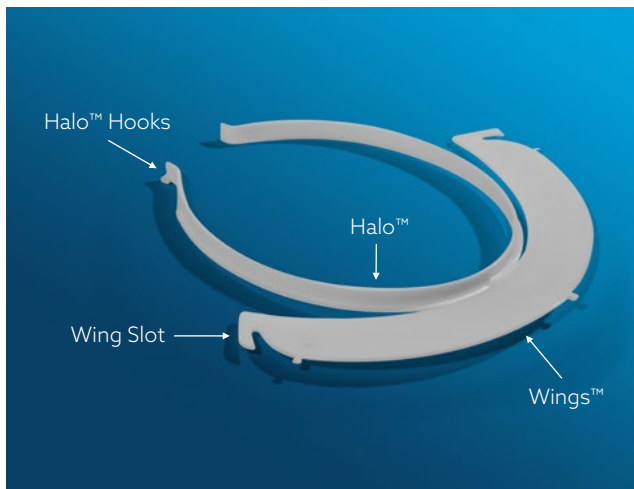

### Assembly

**Step 1:** Hold the frame with the word "Top" facing upwards (found in the center of the Wings). Insert the middle-top hole of the shield into the middle-top stud on the Wings™ and snap into place. (Note: The middle-top hole is the standard position and the middle-bottom hole angles the shield further away from the chin.)

**Step 2:** Holding the Halo™, grab either side of the shield. Insert the slot of the shield into the slot of the Wing™, pull the shield lightly to line up the hole with the stud and snap it on. Repeat for the other side.

### Reordering

Shields and frames can be replaced after repeated use and can be ordered as a kit or separately. Ordering spares will assure uninterrupted protection. Visit [amdrecor.com](http://amdrecor.com) to place your order.

### Adjusting

The Angel Lite™ Frame is designed to be one size fits all and there are different ways to adjust it. Depending on the user's head size, preference, or setting:

- The frame can be slightly bent inward or outward to fit the user's head
- The Wings™ of the frame can rest on, above, or below the lip of the Halo™
- For a tighter fit, place a rubber band or tie on one Halo™ hook and pull it around the back of the head and place on the other Halo™ hook