

Product Features

- Provides full facial protection
- Reusable frame and shield
- Hassle-free and lightweight
- Sturdy and comfortable to wear
- Non-fogging
- Adjustable frame – one size fits all
- Fits over glasses and masks
- Protects against spray or splatter
- NOT to be used for impact



Dimensions

Shield Size: 13.8"W x 9.25"H
Shield Thickness: .012 mm
Frame Size: 7" diameter, 1.0 oz.

Materials Used

Frame: Polystyrene
Shield: PETG

Kit Contains

Angel Max™ Frame
Angel Max™ Reusable Shield

Cleaning & Sterilization Instructions

- Immerse or rinse with soap and warm water to remove any grit
- Dry with a soft cloth (avoid using abrasive cleaners)
- Both the frame and the shield may be disinfected or sterilized using gas, UV, or cold sterilization solutions, which do not leave residue
- DO NOT USE HEAT

FDA Approved Materials

Complies with the OSHA Regulation 29HCF 1910.1030





281.999.2002

info@amdrecor.com
amdrecor.com



Angel Max™

The Angel Max boasts all the benefits of the Angel Lite, but with a slightly thicker shield, and a heavier gauge plastic, allowing this model to stand up to daily, repeatable use and still be remarkably light. Designed to be reusable, just follow our simple cleaning instructions and you'll be able to extend the longevity of this model, which means fewer changeouts and more savings.

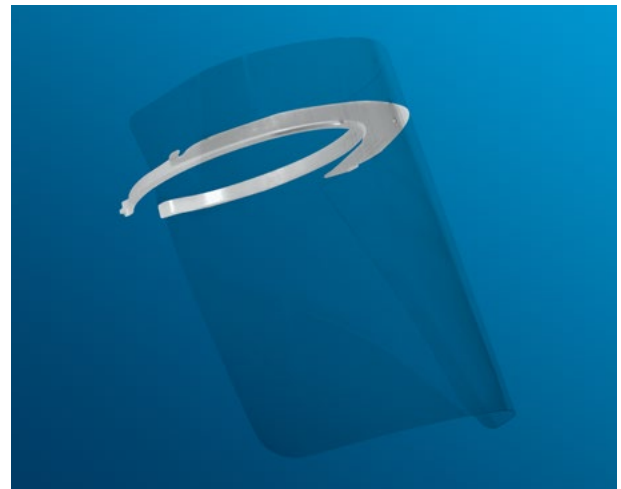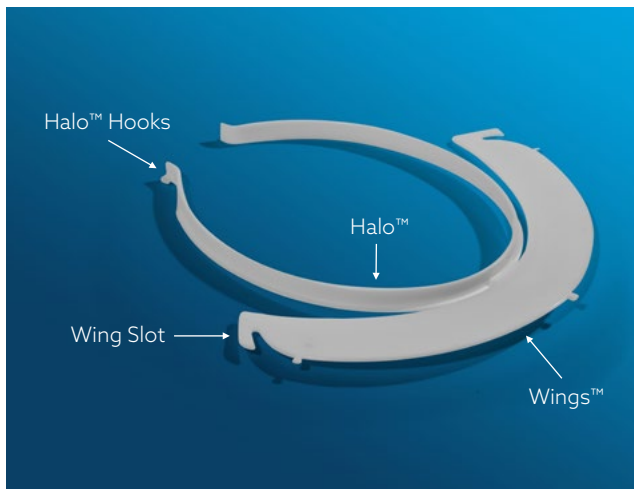

Assembly

Step 1: Hold the frame with the word "Top" facing upwards (found in the center of the Wings). Insert the middle-top hole of the shield into the middle-top stud on the Wings™ and snap into place. (Note: The middle-top hole is the standard position and the middle-bottom hole angles the shield further away from the chin.)

Step 2: Holding the Halo™, grab either side of the shield. Insert the slot of the shield into the slot of the Wing™, pull the shield lightly to line up the hole with the stud and snap it on. Repeat for the other side.

Reordering

Shields and frames can be replaced after repeated use and can be ordered as a kit or separately. Ordering spares will assure uninterrupted protection. Visit amdrecor.com to place your order.

Adjusting

The Angel Max™ Frame is designed to be one size fits all and there are different ways to adjust it. Depending on the user's head size, preference, or setting:

- The frame can be slightly bent inward or outward to fit the user's head
- The Wings™ of the frame can rest on, above, or below the lip of the Halo™
- For a tighter fit, place a rubber band or tie on one Halo™ hook and pull it around the back of the head and place on the other Halo™ hook