

Archangel™

Product Features

- Provides full facial and neck protection
- Reusable frame and shield
- Hassle-free and lightweight
- Sturdy and comfortable to wear
- Non-fogging
- Frame hooks together for easy take on/off
- Large shield for more protection
- Perfect for lights and loupes
- Protects against spray or splatter
- NOT to be used for impact



Dimensions

Shield Size: 12"W x 15.75"H

Shield Thickness: .012 mm

Frame Size: 7.5" diameter, 1.3 oz.

Materials Used

Frame: Polystyrene

Shield: PETG

Kit Contains

Archangel™ Frame

Archangel™ Reusable Shield

Cleaning & Sterilization Instructions

- Immerse or rinse with soap and warm water to remove any grit
- Dry with a soft cloth (avoid using abrasive cleaners)
- Both the frame and the shield may be disinfected or sterilized using gas, UV, or cold sterilization solutions, which do not leave residue
- DO NOT USE HEAT

FDA Approved Materials

Complies with the OSHA Regulation 29HCF 1910.1030





281.999.2002

info@amdrecor.com
amdrecor.com



Archangel™

A unique Amdrecor design intended to remove any contact with the head. The Archangel is a comfortable chest rest face shield meant to be worn for long hours and can accommodate oroscopic telescopes. It protects the face and neck from lower angles without sacrificing airflow over the top. Archangel gives the same lightweight comfort and protection of the Angel Shield Lite and Max, allowing complete freedom of head movement and a full range of motion.

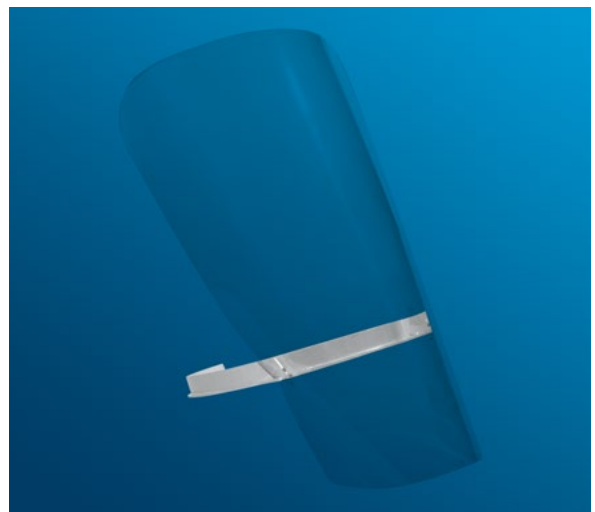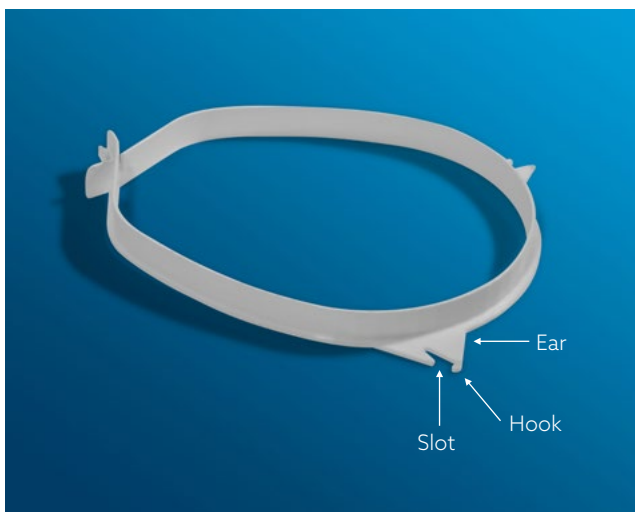

Assembly

Step 1: Hold the frame so that the ears are on the bottom. Insert the shield into the frame slot and secure the hook into the hole of the shield. (Note: The middle hole is the standard position. The top hole angles the shield outward, while the bottom hole angles the shield inward.)

Step 2: Bend the shield around following the frame contour to the other side.

Step 3: Insert the shield into the slot of the other ear and secure the hook into the hole of the shield. Be sure the holes in the shield are securely in the hooks of the frame.

Step 4: Secure the frame around your neck by inserting one notch into the other at the back of the frame.

Reordering

Shields and frames can be replaced after repeated use and can be ordered as a kit or separately. Ordering spares will assure uninterrupted protection. Visit amdrecor.com to place your order.