




## Defender Chest Rest System

### Product Features

- Provides full facial and neck protection
- Reusable frame and shield
- Hassle-free and lightweight
- Sturdy and comfortable to wear
- Non-fogging
- Large shield for more protection
- Perfect for lights and loupes
- Protects against spray or splatter
- Adjustable frame – one size fits all
- NOT to be used for impact

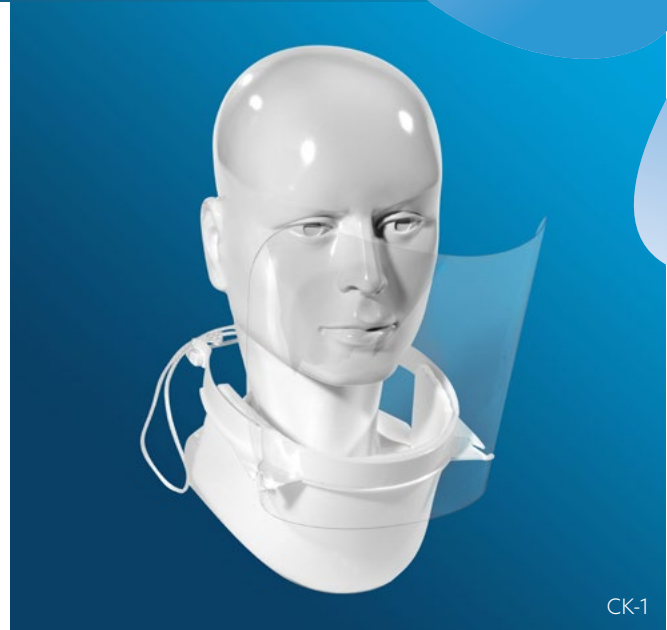

CK-1

### Dimensions

**CK-1 Shield Size:** 11"W x 7.75"H  
**CK-2 Shield Size:** 12"W x 11.75"H  
**Shield Thickness:** .015 mm for both  
**Foam Size:** 11.75"W x 4.5"H x .25"Thick  
**Frame Size:** 7" diameter, 1.0 oz.

### Materials Used

**Frame:** Polystyrene  
**Foam:** Closed Cell  
**Shield:** PETG

### Kit Contains

Defender Frame  
Defender Shield  
Chest Foam  
Chest Tie (String)

### Cleaning & Sterilization Instructions

- Immerse or rinse with soap and warm water to remove any grit
- Dry with a soft cloth (avoid using abrasive cleaners)
- Both the frame and the shield may be disinfected or sterilized using gas, UV, or cold sterilization solutions, which do not leave residue
- DO NOT USE HEAT

### FDA Approved Materials

**Complies with the OSHA Regulation 29HCF 1910.1030**




281.999.2002

info@amdrecor.com  
amdrecor.com



## Defender Chest Rest System

The Defender is the original, unique chest shield on the market. First produced in 1992, this revolutionary chest design allows freedom of head movement without any head pressure points or hair interference. Enjoy protection from the neck area upwards that does not interfere with glasses, masks, or oroscopic telescopes (loupes). A string tie allows for comfort adjustment as well as distance from the face. Two shield sizes available.

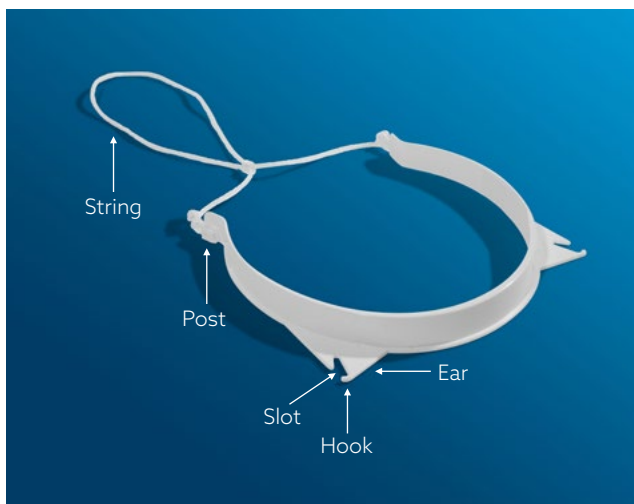

### Assembly

**Step 1:** Peel off the blue protective film from shield.

**Step 2:** Turn frame so ears are on the bottom. Shield is attached by inserting slot of shield into slot of ear. Bend shield to insert hook into hole of shield. Next, bend shield around following the frame contour to the other side to again insert slot of shield into slot of ear, followed by inserting hole of shield into hook.

**Step 3:** Attach foam neck guard and rest to frame in opposite direction of shield. Bend foam to match contour of frame and velcro attachments onto frame. Press velcro firmly to secure.

**Step 4:** Attach one side of the string to the post on the frame and tighten by squeezing loop to post.

### Adjusting

The Defender Frame is designed to be one size fits all and therefore can be adjusted based on user. Adjust the closeness of the shield by shortening or lengthening the cord. To be effective the shield or lens portion must be positioned so that when you are working normally the object in your hands should be viewed a little below the center of the shield. This is normally accomplished when shield is approx. 4-6" from the bridge of the nose and the top of the shield is just below eye level.

### Reordering

Shields and frames can be replaced after repeated use and can be ordered as a kit or separately. Ordering spares will assure uninterrupted protection. Visit [amdrecor.com](http://amdrecor.com) to place your order.