

Product Features

- Provides full facial protection
- Disposable frame and shield
- Hassle-free and lightweight
- Sturdy and comfortable to wear
- Non-fogging
- Opening at top for airflow
- Fits over glasses and masks
- Adjustable frame – one size fits all
- Protects against spray or splatter
- NOT to be used for impact
- Single/limited use



Dimensions

Shield Size: 11"W x 8.75"H

Shield Thickness: .010 mm

Frame Size: 5.5" diameter, 0.4 oz.

Materials Used

Frame: Polystyrene

Shield: PETG

Kit Contains

FeatherLight™ Frame

FeatherLight™ Shield

Cleaning & Sterilization Instructions

- Immerse or rinse with soap and warm water to remove any grit
- Dry with a soft cloth (avoid using abrasive cleaners)
- Both the frame and the shield may be disinfected or sterilized using gas, UV, or cold sterilization solutions, which do not leave residue
- DO NOT USE HEAT

FDA Approved Materials

Complies with the OSHA Regulation 29HCF 1910.1030




281.999.2002

info@amdrecor.com
amdrecor.com



FeatherLight™

Amdrecor's FeatherLight Series face shields are an ultra-light, disposable face shield. This affordable face shield weighs less than an ounce, and is comfortable and cool for work or home use, providing full-facial protection from spray and splatters. Whether it's for single-use or limited multiuse, the FeatherLight's minimal weight will feel like it's not even there.

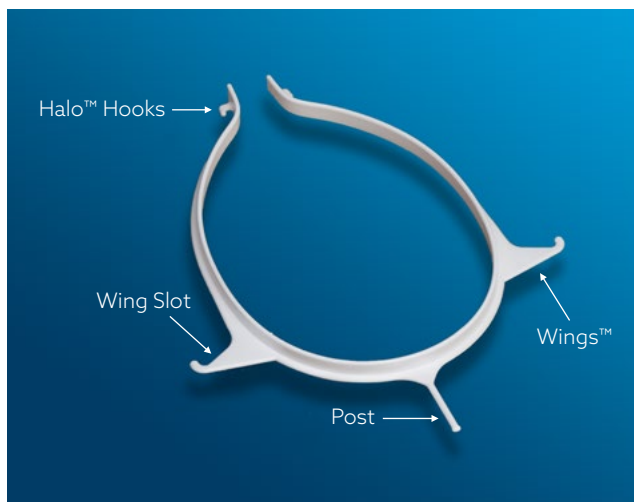

Assembly

Step 1: Align head frame Post and Wings™ with shield holes.

Step 2: Start at either end (right or left) and insert Wings™ into corresponding hole on shield.

Step 3: Move to center Post and insert into top, middle, or bottom hole of shield, depending on desired angle/distance (The angle and distance of the shield to face varies. The top hole is closest to face, the bottom is furthest away).

Step 4: Insert other Wing™ into corresponding hole on shield.

Reordering

Shields and frames can be replaced after repeated use and can be ordered as a kit or separately. Ordering spares will assure uninterrupted protection. Visit amdrecor.com to place your order.

Adjusting

The FeatherLight™ Frame is designed to be one size fits all and there are different ways to adjust it. Depending on the user's head size, preference, or setting:

- The frame can be slightly bent inward or outward to fit the user's head
- For a tighter fit, place a rubber band or tie on one Halo™ hook and pull it around the back of the head and place on the other halo hook