



Protector Reusable Hood

Our reusable hood is for those who enjoy the airflow convenience of our Protector Series but need to close the gap at the forehead. This add-on option simply attaches to any of the Protector Series frames and spans across the top of the shield. Additionally, the reusable hood can be customized with your company's logo.

Assembly

To attach the hood, start by holding it so that the indent is at the top. Insert one side of the hood into the slot of the frame and secure the hole over the hook. Next, bend the hood around, following the frame contour, to the other side. Then, insert the hood into the other slot of the frame and secure the hole over the hook.



Dimensions

Hood: 10.5"W x 4.25"H
Thickness: .015 mm
Weight: 0.4 oz.

Materials Used

Frame: Polystyrene
Shield: PETG

Contains

Reusable Hood for Protector Series

Cleaning & Sterilization Instructions

- Immerse or rinse with soap and warm water to remove any grit
- Dry with a soft cloth (avoid using abrasive cleaners)
- May be disinfected or sterilized using gas, UV, or cold sterilization solutions, which do not leave residue
- DO NOT USE HEAT

FDA Approved Materials

Complies with the OSHA Regulation 29HCF 1910.1030



281.999.2002

info@amdrecor.com
amdrecor.com

