


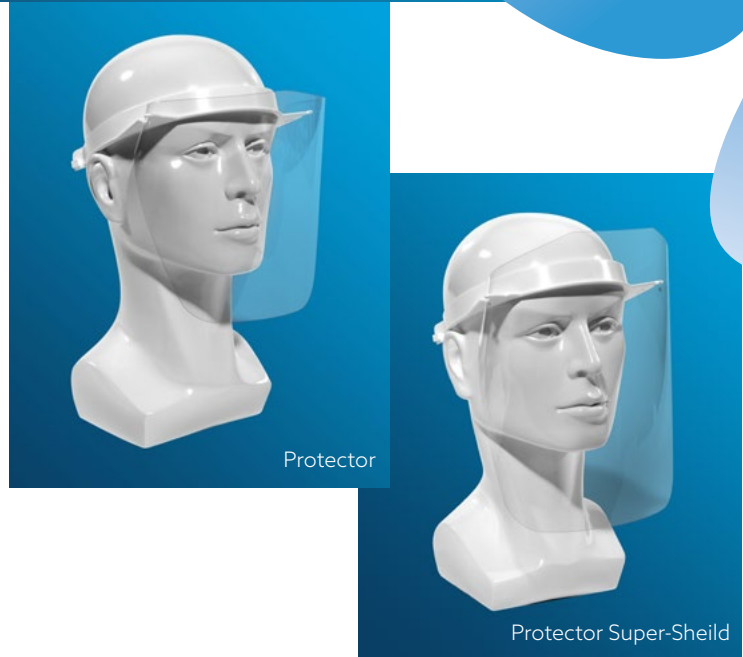

Protector and Protector Super-Shield

Product Features

- Provides full facial protection
- Reusable frame and shield
- Hassle-free and lightweight
- Adjustable frame – one size fits all
- Adjustable shield angles
- Sturdy and comfortable to wear
- Fits over glasses and masks
- Non-fogging
- Protects against spray or splatter
- NOT to be used for impact



Dimensions

Protector Shield: 11.25"W x 7.5"H

Protector Super-Shield: 11"W x 9.75"H

Shield Thickness: .015 mm for both

Frame Size: 7" diameter, 1.0 oz.

Materials Used

Frame: Polystyrene

Shield: PETG

Kit Contains

Protector Frame

Protector Shield

Cleaning & Sterilization Instructions

- Immerse or rinse with soap and warm water to remove any grit
- Dry with a soft cloth (avoid using abrasive cleaners)
- Both the frame and the shield may be disinfected or sterilized using gas, UV, or cold sterilization solutions, which do not leave residue
- DO NOT USE HEAT

FDA Approved Materials

Complies with the OSHA Regulation 29HCF 1910.1030



281.999.2002

info@amdrecor.com


amdrecor.com





Protector and Protector Super-Shield

Our Protector Series face shields are the original models that got us where we are today. While these precede the Angel Shield, they provide the same frontal protection. Open at the top to provide more airflow, these lightweight facial shields protect from infection-causing sprays, blood-borne pathogens, and splatters. The Protector Super Shield provides maximum protection via a larger shield than the Protector model.

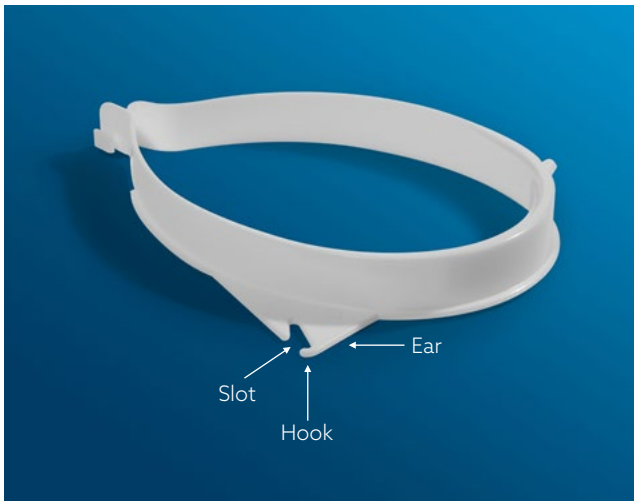

Assembly

Step 1: Peel off the blue protective film from shield.

Step 2: Hold the frame so that the ears are on the bottom. Insert the shield into the frame slot and secure the hook into the hole of the shield.

Step 3: Bend the shield around following the frame contour to the other side.

Step 4: Insert the shield into the slot of the other ear and secure the hook into the hole of the shield. Be sure the holes in the shield are securely in the hooks of the frame.

Reordering

Shields and frames can be replaced after repeated use and can be ordered as a kit or separately. Ordering spares will assure uninterrupted protection. Visit amdrecor.com to place your order.

Adjusting

The Protector Frame is designed to be one size fits all and therefore should fit most heads. However, the head-frame can be opened or closed for a more comfortable fit. Under normal circumstances, this may be accomplished simply by over-opening or over-closing as needed. The headframe is pliable and should not break.

The shield angle is also adjustable. For the shield to be closer to the chin, use the top most hole. For a more relaxed shield distance from the chin, adjust using the lower holes.